



E-Pacer

The Rifton E-Pacer combines a powerful electric lift column and trunk support with the key gait training features of the classic Pacer. Its sit-to-stand functionality removes a major barrier to gait training for larger or highly dependent clients. The optional thigh straps provide seat-to-seat transfer capability for added versatility.

- Zero lift sit-to-stand and seat-to-seat transfers
- Expandable base frame
- Accommodates users up to 6'5" and 350 lb



Scan here to order, request a quote, or see detailed product information.



Key features

1 Zero lift transfers

The E-Pacer can perform sit-to-stand transfers as well as seat-to-seat transfers with no lifting on the part of the caregiver.



2 Expandable legs

Widen the base legs for clients needing extra step width or to get up close to a wheelchair prior to a transfer.



3 Height range

The 26" adjustment range of the E-Pacer's electric lift column provides versatility for use with clients of different statures.



4 Built-in scale

Optional Bluetooth scale and Gait Tracker app allow caregivers to track weight-bearing progress over time.





Scan for sizing
and pricing.



E-Pacer components



Scale

Weigh clients during transfers or measure weight-bearing during gait training. Pair via Bluetooth with a phone to track progress using Rifton's Gait Tracker app.

Note: The scale must be included in the original purchase. It cannot be added later.



Odometer

No more counting tiles! The odometer easily records walking distance in feet or meters.



Front handle

Allows caregivers to maneuver and guide the E-Pacer, with up/down switch for body support height adjustment.



Switch pole

Lets caregivers operate the electric lift without bending down. Reversible design gives the client control when desired.



Arm prompts

Height, rotation, angle, and positioning adjustments.



Arm platforms

A simple arm prompt alternative for positioning.

Padded surface dimensions: 13½" x 5½"



Hip positioner

Encourages proper positioning and natural movement during gait training. Provides abduction support with optional comfort pad.



Pelvic support

Available in three sizes, the pelvic support assists with sit-to-stand transfers, gait training, and fall prevention—ideal for early mobility recovery.



Thigh straps

For seated transfers—machine washable and available in narrow or wide styles.



Ankle prompts

Prevent scissoring and control stride length with padded straps and spring adjusters.



Accessories tote

Keeps all E-Pacer components together. Large, durable, and attractive design.

Need support?

The Rifton website offers video tutorials, training resources, sample letters of medical necessity, evidence-based articles, and more.



Scan here to order, request a quote, or see detailed product information.

Frame color options



Red



Blue



Black



Lime



Pink

