


Silvalea® slings laundry instructions

Before washing any sling and in order to protect the straps from being damaged:

- Make sure any Velcro or buckles are fastened together securely.
- Remove the plastic ribs from any models that contain these.


General stand aid with loops:
quilted polyester

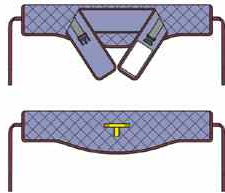

Wash at 85°C
Tumble dry low heat
(max 50°C)




General stand aid with clips:
quilted polyester

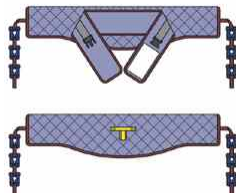

Wash at 85°C
Tumble dry low heat
(max 50°C)




Fastfit padded leg sling: mesh

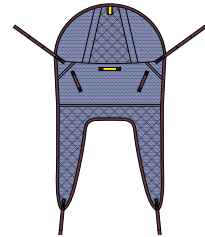

Wash at 85°C
Tumble dry low heat
(max 50°C)




High easy sling with clips: mesh

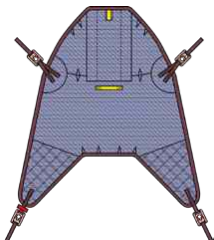

Wash at 85°C
Tumble dry low heat
(max 50°C)




Universal deluxe sling with head
support: open weave mesh

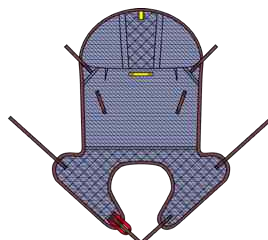

Wash at 85°C
Tumble dry low heat
(max 50°C)




Silva-Supreme general purpose
bariatric sling: heavy duty polyester

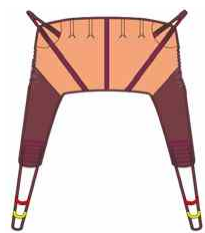

Wash at 85°C
Tumble dry low heat
(max 50°C)




Toileting sling: mesh

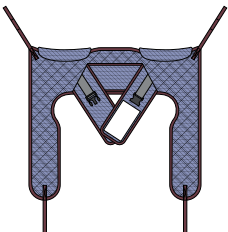

Wash at 85°C
Tumble dry low heat
(max 50°C)




Disposable sling with loops

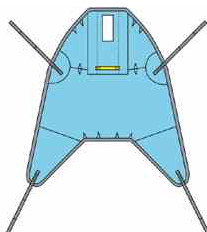

Designed for short term use.
Not to be laundered.
A 'DO NOT USE' label will become
visible when in contact with liquid.

Disposable sling with clips

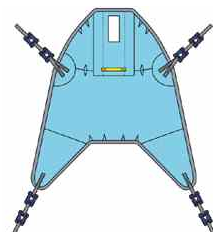

Designed for short term use.
Not to be laundered.
A 'DO NOT USE' label will become
visible when in contact with liquid.

Cleaning your slings is a great opportunity to make sure they are safe and ready for action.
Check out our handy sling safety checklist in the resource section of cubro.co.nz