

Positioning Checklist for the Rifton HTS

Use this Positioning Checklist as a convenient way to ensure optimal use of the HTS. Write notes to customize your instructions for each individual.

INDIVIDUAL'S NAME _____

STAFF NAME _____

DATE _____



THIS POSITIONING CHECKLIST IS INTENDED FOR CONSISTENT USE OF ADAPTIVE EQUIPMENT FOLLOWING ASSESSMENT BY A HEALTH PROFESSIONAL. THE CHECKLIST DOES NOT REPLACE AN INITIAL CLINICAL EVALUATION FOR CORRECT SIZING, COMPONENT SELECTION, AND APPROPRIATE THERAPEUTIC POSITIONING FOR THE INDIVIDUAL. SOME COMPONENT SIZING IS DEPENDENT ON THE BASE MODEL SIZE.

Base and size

☐ Mobile tilt-in-space with footboard

- ☐ small
- ☐ medium
- ☐ large



☐ Stationary non-tilt with footboard

- ☐ small
- ☐ medium
- ☐ large



☐ Mobile non-tilt with footboard


- ☐ small
- ☐ medium
- ☐ large



☐ Mounting bar for elongated toilet



☐ Mounting bar for round front toilet

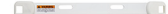

Placement for use

- ☐ With pan
- ☐ Over toilet
- ☐ With bowl adapter



On



Over



Off

☐ Seat pad

- ☐ Rear-closed seat pad
- ☐ Rear-open seat pad



Seat height (from floor)

Specify number of notches visible on chair frame leg _____

Seat depth

Inches (cm) _____

Seat width

☐ Standard hip guides ☐ Thick hip guides

Inches (cm) _____

Seat back angle

- ☐ Forward position (6° anterior; recommended for toileting)
- ☐ Center position (6° posterior)
- ☐ Posterior position (18° posterior)



☐ Footrest



Footrest height: Specify number of notches visible on footrest upright bar: _____

Footrest angle

- ☐ 30° forward ☐ 20° forward ☐ 10° forward ☐ 10° back
- ☐ Calfrest
- ☐ Ankle straps
- ☐ Left foot ☐ Right foot

☐ Anterior support/tray



☐ Abductor



☐ Deflector

☐ Standard Deflector

☐ Splash guard



Standard



Splash guard

☐ Trunk laterals measure in inches (cm)

Width apart: _____

Height of right lateral (measure from seat to top of support): _____

Height of left lateral (measure from seat to top of support): _____

With chest strap?

☐ Yes ☐ No



Laterals



Laterals with chest strap

☐ Butterfly harness



Placement of upper straps prior to securing the buckle

☐ Over the top of the backrest

☐ Through the top slots of the backrest

☐ Through the second slot down from top of backrest

☐ Through the third slot down from top of backrest

☐ Headrest measure in inches (cm)

Height of headrest (measure from seat to top of headrest) _____

